

BUSCOT BABEVAC COT ^{CE}

(Part of the "SKI" range of products)

Designed for the safe evacuation of premature and new born babies from maternity wards and special care baby units in the event of fire or similar emergency.

Features:

- Can carry up to two babies depending on size and condition.
- Enables carers to be 'hands free' to administer needed care to the infant while evacuating.
- Secure storage for one size CD oxygen cylinder on the underside of the cot to allow manual ventilation of baby if required (Cylinder not included).
- Flexible strong construction allows for easy handling and adequate support and protection of babies.
- Adjustable shoulder straps.
- Wall mountable for easy, instant access.
- Easy wipe clean after use.



Using the Buscot Babevac Cot:

- Remove the Buscot from its storage bag and unfold.
- Ensure that the cot is on a flat surface.
- Swaddle up the baby/ babies for warmth and place inside the cot.
- Place the straps over the shoulder - creating a single cross over the chest area, and fasten using the plastic safety buckles.
- If manual ventilation is needed, a second carer will need to secure the cylinder to the underside of the Buscot.
- Move the baby/ babies within the Buscot, to a safe area.

Specifications:

Packed: 62cm x 36cm x 9cm

Opened: 58cm x 30cm x 28cm

Padding: 5mm

Weight: 2kg

Material: Vinyl

Cylinder capacity: Size CD (not included)

Product code: BUS001



HOSPITAL AIDS
Evacuation Equipment Specialists

1 Kingsthorpe Business Centre, Tel: +44(0)1604 586501
63 Studland Road, Kingsthorpe, Fax: +44(0)1604 586576
Northampton, NN2 6NE. England sales@hospitalaids.co.uk
Registered in England No 1777125 www.hospitalaids.co.uk



MADE IN THE UK
DISTRIBUTED WORLDWIDE