

# SKI PAD<sup>CE</sup>

(Part of the “SKI” range of products)


The Ski Pad is designed for use in places where space is limited, such as congested hospital wards and buildings with narrow corridors or single width fire doors.

## Features:

- Durable vinyl coated base material enables the Ski Pad to move over virtually any interior floor surface.
- 5cm foam pad provides protection for the user.
- Seat belt style metal buckles and webbing straps.
- Foot pocket at each end for ease of use.
- Wall mountable, wipe clean bag.

## Using the SKI PAD:

- In an Emergency the Ski Pad is quickly removed from its wall mountable storage bag.
- The patient can then be transferred on to the Ski Pad which can be performed either from a bed, floor or chair.
- Place feet within the foot pocket.
- If pillows are available place these on the individual's leg and chest area (for comfort once strapped in).
- Secure the individual in place on the pad using the seat belt style cross straps.
- Using the end pulling handles the individual can be moved to a safe area. When going down stairs it is important to pull the individual feet first.

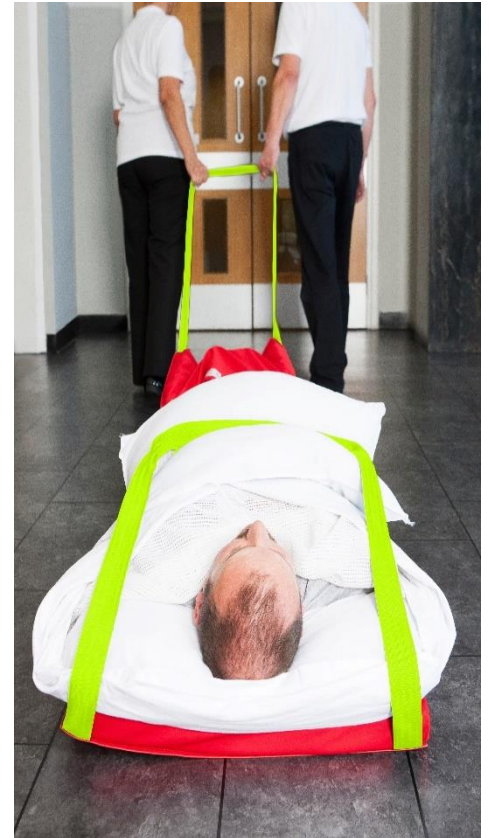

## Specifications:

Packed: 67cm x 64cm x 20cm (2 or 3 strap)  
 Unrolled: 197cm x 61cm  
 Padding: 5cm thick - Weight: 5kg  
 Safety tested to: 160kg  
 Safe working load: 120kg – for two people  
 Material: Cotton (top) - Vinyl (bottom)



## Product codes:

Two Strap: SKIP01  
 Three Strap: SKIP02

Onsite demonstrations and training packages are available contact us for more information

**HA** **HOSPITAL AIDS**  
 Evacuation Equipment Specialists  
 visit: [www.hospitalaids.co.uk](http://www.hospitalaids.co.uk)

Tel: +44 (0)1604 586501 [info@hospitalaids.co.uk](mailto:info@hospitalaids.co.uk)



MADE IN THE UK  
 DISTRIBUTED WORLDWIDE